Pupilómetro Digital PD-100

Digital PD Meter PD-100

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USER'S GUIDE

CE
0434

3B OPTIC INSTRUMENTS

C O N T E N I D O S

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C O N T E N T S

Introduction	Introduction	10
Symbols information	Symbology	10
Warnings & Cautions	Warnings & Cautions	11
Components	On/Off Button	11
	Infrared Communication Port	11
	Measurement Slide Buttons	11
	Distance Selecting Buttons	11
	Operator's Window	12
	Print Button	12
	Hold Button	12
	Digital Display	12
	Forehead Rest	12
	Lens Power Corrector	12
	Eye Selector	12
	Patient's Window	12
	Nose Support	12
	Battery Lid	12
Special Characteristics	Fixation LED - Light level	12
	Selecting light level	12
	Automatic Shut-Off	13
	Low Voltage Indicator	13
Instructions for Use	Introduction	13
	PD-100 Operation	13
	Adjustable Viewing Distance	13
	Hold Measurement Data	13
Cleaning	Procedures	14
Maintenance	Procedures	14
	Battery Change	14
Troubleshooting	Troubleshooting Chart	14
Product Specifications	Specifications	15
Operating conditions	environmental operation, transportation and storage conditions	15
Additional Information	Ordering Information	16
Warranty	Warranty	16
	Product Changes	16
	Claims for damages in transit	16

Introduction

Congratulations for purchasing the new PD-100 Digital Pupillometer.

This User's Guide is a training and reference manual. In order to ensure the optimum performance from your PD-100 Digital Pupillometer, we recommend that you read carefully and follow the steps in this guide. It is also important that you keep this guide for future reference and to share with other users. For additional copies or assistance, please contact:

3B Optic Instruments
Ocampo 370 – 2000 – Rosario – Santa Fe – Argentina
Tel.: 0341-4820414 / Fax: 0341-4818334
E-mail: ventas@3boptic.com - Depto. Técnico: 3bopticst@gmail.com

Medes Ltd.:
5 Beaumont Gate, Shenley Hill, Radlett, Herts WD7 7AR, London - England.
Tel/Fax: +44 1923859810

Symbols information:

Symbology

Important instructions for maintenance and operation are included in this User Guide symbol

Caution symbol indicating important information included in this User Guide

Warning symbol indicating important information and maintenance instructions that are included in this User Guide

Manufacturer data

Year of manufacture

Serial number

Dispose of as electrical or electronic equipment

Approved with the Medical Device Directive 93/42 / EEC

Fragile contents, handle with care

Keep dry. The package must be away from rain

Authorized representative in the European Community

This side up. Indicates the correct position of the package

Do not stack more than 5 boxes

Rate applicable Part B

©2017

All rights reserved.

No part of the present publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, recording or otherwise, without the prior written authorization of the manufacturer.

Warnings and Cautions

! Caution: This instrument must be used by especially trained personnel.
! Caution: Read this user guide before using this instrument.
! Caution: Do not use the PD-100 in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
! Caution: Do not dispose of the device or any of its accessories with house waste. Check local regulations for proper disposal. It's responsibility of the user handed over to a designated collection point for recycling of electrical and electronic or contact the manufacturer or the authorized representative of the manufacturer to proceed with disposal in a safe and environmentally friendly manner appliance.

! Warning: Keep the PD-100 in a cool, dry, dust-free environment.
! Warning: Make sure the patient's window is always clean; you may use a lens tissue or a clean, dry soft cloth to clean this window.
! Warning: Do not use alcohol or other strong cleaning solutions to clean the outer surface, windows or screen of this instrument or damage to these parts may occur.
! Warning: Try not to hit or get whet this instrument.
! Warning: After using this instrument keep it in the original pack.
! Warning: Do not use this instrument with patients that present ulcers in the area of contact with the equipment.
The manufacturer is not responsible for the safety and reliability of this instrument when:
• The instrument is assembled, disassembled, repaired or modified by unauthorized dealers or persons.
• The instrument is not used in accordance to this User's Guide.

Components

On/Off Button

This button turns the unit on or off. When pressed on, the instrument starts a self – test routine. Once this routine has been completed, the unit is ready for operation.

Infrared Communication Port

By means of this Infrared Communication Port the measurement data can be immediately transferred to a printer or personal computer without using a standard printer cable.

Measurement Slide Buttons

The Measurement Slide Buttons are pressed to adjust the position of the Index Lines for accurate measurement of the interpupillary distance (PD) and Right/Left Nose-to-Pupil Distance (R/L NPD). During the measurement process, each measurement slide is positioned so that the index line is over the center of the corneal reflection light at the center of the patient's pupil.

Note: When one of the measurement slides is moved too far to the left or to the right by pressing the respective button, the Digital Display will show "00,o" on that side of the screen, indicating that it must be repositioned within the span signal.

Distance Selecting Buttons

These buttons are used for modifying the measurement data to reflect short and medium fixation distances. When measurement data is obtained, they are set to infinity (∞), and then adjusted to display the short and middle range data.

Components (cont.)

Operator's Window

This is the window through which the operator looks at the patient's eye.

Print Button

This button sends the measurement data out of the Infrared Communication Port.

Hold Button

By pressing this button the current measurement data is retained on the Digital Display, not allowing it to change even if the Measurement Slide Buttons or the Distance Selecting Buttons are pressed. If pressed a second time the data is released and can be modified.

Digital Liquid Crystal Display

The PD 100's improved LCD screen allows a quick and clear view of the measurement data and operation symbols.

Forehead Rest

The Forehead Rest provides the instrument with a better stability, allowing for more reliable and accurate measurement data.

Lens Power Selector

By means of its two levels of ocular power for enhancing the operator's view of the pupil, the PD 100 provides accurate inter pupillary distance measurements.

Eye Selector

This feature enables the occlusion of the fixation LED from the patient's eyes. The Eye Selector has three positions. The right and left positions which occlude the fixation LED from one eye (right or left), and the middle position which allows the patient to see the fixation LED with both eyes.

Patient's Window

This is the window through which the patient looks during the measurement process.

Nose Support

This feature avoids problems of side movements, allowing for more accurate measurements.

Battery Lid

This lid provides access to the battery.

Special Characteristics

Fixation LED

The Fixation LED (Light Emitting Diode) inside the PD-100 is a visual target for the patient during the measurement process. This target has a circular shape and is continuously on during the measurement. If the Measurement Slide Buttons or the Distance Selecting Buttons are not pressed for approx. 20 seconds the LED will automatically turn off. It can be turned on again by pressing the Measurement Slide Buttons or the Distance Selecting Buttons.

Selecting light level

This feature will allow the user to select 3 different levels according to his own visual comfort. In order to enter the light calibration mode follow this instructions:

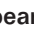
Turn on the equipment and keep the PRINT and HOLD buttons pressed for only two seconds at the same time. The display will read , for example: L H 2 (level 2)

That is to say, you will see the light level that had been previously chosen, as follows: Level 1; Level 2 (default); Level 3, being Level 1 the least intense and Level 3 the most intense. By pressing the PRINT button you will be able to choose the intensity according to your comfort. To store your choice in the memory press HOLD.

Automatic Shut-Off

The electronic circuits of the PD-100 are designed for conservation of battery power. After 20 seconds of inactivity the Fixation LED shuts off. If the inactivity of the instrument continues for a total of 1 minute and 20 seconds the unit turns completely off.

Low Voltage Indicator

The PD 100 is provided with a Low Voltage indicator circuit for monitoring the voltage of the battery. A special icon () appears on the LCD display indicating the voltage level. This special feature allows the operator to change the battery in time, thus preventing incorrect interpupillary distance measurements caused by low voltage conditions.

Instructions for use

Introduction

The Breitfeld & Schliekert PD-100 Digital Pupillometer provides the latest measurement technology for calculating interpupillary distance and right/left nose-to-pupil distance for short, middle and long range viewing distances.

PD-100 Operation

To operate the PD-100 perform the following steps:

1. Press the On/Off Button. The PD-100 starts a self-test and a full display of the digits appears on the screen as it checks the operation of the instrument. Once the self-test has been successfully completed, the Display will show the following pupillary distances: 31,0 – 62,0 – 31,0
2. Make sure that the Distance is set to infinity (∞).
3. Place the Forehead Rest gently against the patient's forehead and set the Nose Support onto the patient's nose.
4. Set the Eye Selector to either right or left position and tell the patient to look at the circular fixation light with that eye.
5. Look through the Operator's Window and observe the corneal reflected light on the center of the patient's cornea. Then, press the Measurement Slide Buttons until the index line is directly over the center of the corneal reflected light.
6. Set the Eye Selector to the opposite position and repeat the above procedure.
7. Observe the Digital Display and check the measurements. The right and left measurements should be approximately equal, and the middle measurement should be the sum of both measurements.

Adjustable Viewing Distance

After the right and left measurements are taken, the measurements for medium and short ranges for the viewing distances can be obtained by pressing the Distance Selecting Buttons.

Note: If the Hold Button is pressed before the medium and short viewing distances are obtained, the measurement data on the Digital Display will not change to reflect the shorter viewing distances.

Hold Measurement Data

Once the measurement data is obtained, the Hold Button may be pressed to prevent the data on the Digital Display from changing if the Measurement Slide Button or the Distance Selecting Buttons are pressed accidentally. To suppress the hold measurement data mode, press the Hold Button a second time.

Cleaning

Clean the external surfaces of the PD-100 with a mild soap solution. Perform the following steps:

Note: Do not use alcohol or other strong cleaning solutions to clean the outer surface, windows or screen of this instrument or damage to these parts may occur.

Note: Do not immerse the PD-100 in liquid or damage to the instrument may occur.

1- Use a soft clean cloth moistened with a mild soap solution and wipe the external surface of this instrument.

2- If necessary, remove any residual soap solution using a moistened clean soft cloth.

3- Use a clean , dry soft cloth to dry the exterior of the PD-100.

Maintenance

The expected shelf product, based on the availability of spare parts, life has been set at 10 years from the date of manufacture. If the equipment is properly used in accordance with the instructions in the user's guide, preventing bumps or rough handling and keeping it in its original case, it will not need servicing nor calibrating in its lifetime.

Procedure

When the unit indicates low voltage by displaying a special icon () on the Digital Display or the battery is rundown and the unit does not turn on, perform the following steps:

Battery Change

1- Remove the battery lid.

2- Remove the battery and replace it with a new battery.

3- Put the battery lid into its original position.

This is the only maintenance procedure that can be performed by the user on the PD-100.

For all other maintenance-related inquiries refer to:

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Troubleshooting

PROBLEM	SOLUTION
PD-100 does not turn on.	Battery is discharged or missing. Replace battery.
Circular Fixation LED does not stay on.	PD-100 has not been used for 20 seconds. Press the Measurement Slide Buttons or the Distance Selecting Buttons.
Data does not change when the Measurement Slide Buttons or the Distance Selecting Buttons are pressed.	The Hold Measurement Data Mode is on. Press the Hold Button again.
Zeros appear on the Digital Display.	The Measurement Slides are out of range.
If inaccurate measurement is observed.	Contact authorized dealer.

Product Specifications

Display	Liquid Crystal Display
Fixation	Illuminated Circle
Fixation Lamp	Green LED 3 Levels
Occlusion	Right or Left Eye
Battery Type	9 V Alkaline
Measurement:	
Binocular	46,0 83,0 mm \pm 0,5 mm
Monocular	23,0 41,5 mm \pm 0,5 mm
Distance	30 mm - ∞ inf.
Steps	30 35 40 50 65 cm, 1 m, 2 m, ∞ inf.
Dimensions:	
Height	1.96 in. (50 mm)
Width	6.37 in. (162 mm)
Depth	9.72 in. (247 mm)
Color	Grey
Weight	1.41 lbs. (570g.)
Software Version	010603
Applicable part	Type B / Nasal support
Energy source	Device with internal power source
Operating mode	Continuous
Degree of protection harmful ingress of water and particles	IPX0
Sterilization	It does not require sterilization. • Device unsuitable for use in the presence of flammable anesthetic mixture with air or oxygen or oxide Nitrous. • Not intended for use in environments enriched with oxygen

Operating conditions:

Environmental:	
The environmental conditions of use are:	
Operation:	
Ambient temperature	5 °C - 50 °C
Relative humidity (non-condensing)	10 - 90 %
Atmospheric Pressure	600 - 1060 hPa
Transport and storage:	
Ambient temperature	-10 °C - 50 °C
Relative humidity (non-condensing)	10 - 90 %
Atmospheric Pressure	500 - 1060 hPa

The specifications and design are subject to change for improvement without previous notice.

Ordering Information

The supplier, upon request, must provide circuit diagrams, component parts lists, descriptions, calibration instructions or other information that is useful to properly trained technicians to repair any parts that are described by the manufacturer as repairable. For additional copies of this manual, further related information or a list of our authorized dealers, contact 3B Optic Instruments.

Warranty

This product is warranted by 3B Optic Instruments for a period of one year from the date of invoice to the original purchaser against faulty material or poor workmanship provided the instrument is used in accordance with operating instructions. (Note: Authorized dealers should not be considered original purchasers).

3B Optic Instruments' liability under this warranty is limited to replacing or repairing the defective part or product at 3B Optic Instruments discretion.

This warranty applies to new products and does not apply to products which have been altered in any way, damaged due to improper use, inadequate maintenance, disregard of operating instructions or which have the serial number removed, altered or effaced. Nor shall this warranty be extended to products which have been sold, serviced, installed or repaired by unauthorized dealers or technicians.

Lamps, bulbs, batteries, charts, cards and other non-durable items shall not fall within the scope of this warranty.

All claims under this warranty must be made in writing and directed to 3B Optic Instruments or to the authorized dealer who made the original sale and must be accompanied by a copy of the purchaser's invoice.

This warranty is in lieu of all other warranties implied or expressed. No representatives nor other person is authorized to make any other obligations for 3B Optic Instruments. 3B Optic Instruments shall not be liable for any special, incidental, or consequent damages for any negligence, breach of warranty, strict liability or any other damages resulting from or relating to design, manufacture, sale, use or handling of the product.

Product Changes

3B Optic Instruments reserves the right to make changes in design or to make additions to or improvements in its products without obligation to add those changes to the units previously manufactured.

Claims for damages in transit

3B Optic Instruments shipping responsibility ceases with the safe delivery in good condition to the transportation company. Claims for loss or damage in transit should be made directly and immediately to the transportation company.

If, upon delivery, the outside of the packing case shows evidence of rough handling or damage, a "Received in Bad Order" notation on the delivery receipt should be requested to the transportation Company's agent. If within 48 hours of delivery, concealed damage is noted upon unpacking the shipment and no exterior evidence of rough handling is apparent, the transportation company should be requested to make a "Bad Order" report. This procedure is essential for the dealer to maintain the right of recovery from the carrier.